Portable Electric Breast Pump

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0%

BPA

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
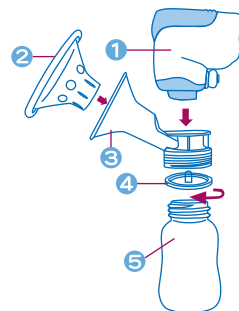
- Put (2) Breast Shield Insert into (3) Breast Shield and connect them tightly.
- Put (4) Rubber into the screw thread of (3) Breast Shield.
- Insert vertically (1) Main Body into (3) Breast Shield and connect them tightly.

- Tighten upward the combined Main Body, Breast Shield and (5) Milk Bottle by clockwise direction.
- * Before use, please pay attention to the correct combination of all parts.

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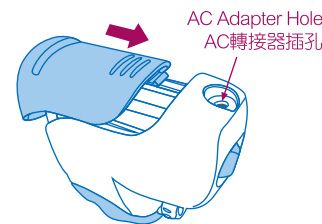- Open the battery cartridge cover.
- Use 2 pcs of 3 AA batteries (This product is limited to use alkaline batteries only.)
- Install the batteries ~ make (+) positive and (-) negative of the battery consistent with the marks in the battery cartridge.
- Put back the battery cartridge cover.
- If this product will not be used for a long time, please remove the batteries.
- When the AC Adapter is used, please insert the plug (DC3V) into the front jack on the top of main body. (Note: please use the AC Adapter dedicated for Breast Pump. If use the other adapter, it might cause breakdown or unpredictable danger. Please don't use it.)

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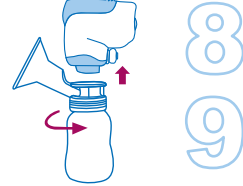
* Suggest use AC Adapter as the power supply of this product for optimal effect.

1. Put Breast Shield Insert into Breast Shield and connect them tightly. (The uplift design of Breast Shield Insert is to promote milk secretion.)
2. Tighten Breast Shield and Milk Bottle firmly.
3. Let Breast Shield be close to the breast slightly and make Teat located in the middle of Breast Shield Insert.
4. Sucking Method : Slip the power switch to ON (with continuous sucking function) to start Breast Pump.
5. The Suction Regulator is to adjust the sucking force while sucking. To enhance, please turn it clockwise; to weaken, just turn anti-clockwise.
6. After getting the proper sucking force, press the Push button slightly for 2~3 seconds and then release the button to motivate the suction, so that it will simulate baby's sucking action and even massage your breast for smoother sucking. Repeat this action until the required milk volume. (Use another hand to massage breast at the same time for better sucking effect.)
7. After finish collecting the breast milk, press the Push button to release the sucking force and turn off the switch.

8. Push out the main body vertically and carefully (when take it out, do not turn it left and right.)
9. Turn anti-clockwise to remove the Breast Shield. Take out Breast Shield Insert. Put Teat into Bottle Cover and connect to the Milk Bottle by clockwise direction, so that the Milk Bottle can be used to feed baby.

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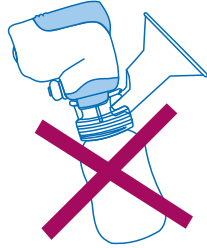
Before the initial use and after each use, wash all of the parts by mild detergent or warm water (except for the main body and battery cartridge). The Nipple, Milk Bottle, and Breast Shield Insert can be sterilized in a Sterilizer (except for the Main Body).

The way to clean the Main Body : After each use, suggest do the maintenance according to the following to procedures to prevent any possible residue milk in the main body and motor parts.

- 1.Disassemble the Main Body from the milk bottle.
- 2.Install the Pump Cleaning Connector below the Main Body.
- 3.Dip the Cleaning Connector into small warm water container about 2 cm.
- 4.Put Pump Cleaning Connector into the container to let Pump Vent be outside the container. (This way is very important to prevent the dirty water draining into the clean water in the container.) If use AC Adapter, make sure AC Adapter can not contact with water.
- 5.When clean the Main Body, open the power switch. After the cleaning water flowing from Pump Vent, draw out the Main Body to make motor turn idly for about 30 seconds to remove the water inside the Main Body.
- 6.Use paper tissue to wipe the Main Body cleanly to avoid the entry of ants and preserve it well for use next time.

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- 1.When use 3 AA alkaline batteries, if LED light vanishes or sucking force is not enough, it means the shortage of electricity. Please replace a new set of batteries to return to normal function.
- 2.When use it, make sure the main body is fixed in the vertical position. If the main body tilts, the breast milk will enter into motor and battery cartridge to cause damage; in case it does, please turn off the power immediately and take out the pump and battery cartridge, and start the pump to let motor rotate for 2~3 minutes and then turn off the motor. Throw the milk inside the motor and battery cartridge out and then clean the battery cartridge by clean cloth.
3. Do not dip the main body, motor and battery cartridge into the water nor wash with water.

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* Unable to start it :

- 1.Check the position of battery polarity is correct.
- 2.Check the connected point of battery is connected tightly.
- 3.Use AC Adapter and make sure the connector of adapter and wiring plug are inserted correctly.

* Unable to pump milk :

- 1.Make sure all of the parts are correctly fastened and connected completely.
- 2.Check the power of battery is normal.

- 3.Turn the motor for 30 seconds at least to obtain the maximum sucking force. (Note: When the battery power is attenuating, the sucking force will weaken.)
- 4.Check the adjusting knob is turned to the state of no suction.

* The sucking force is not enough:

- 1.When LED light will not flash, it means the power is not enough. Then please replace a new set of alkaline batteries. Or use AC Adapter to improve the sucking force of pump.
- 2.Check Breast Shield and breast are close enough.

■ 吸乳要點及注意事項/Pumping Instruction and Attention

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1. After giving birth, if the breasts supply low milk and you need to use the breastpump, please consult your lactation consultant or doctor for directions.
2. For the case of mastitis, inverted or flat nipples, please consult your doctor or lactation consultant for dilrections.
- 3.Do not use this product if milk cannot be expressed with hand. Please consult your doctor or lactation consultant for directions.
4. Breast massage or cover the breasts with a warm cloth to encourage the milk supply before pumping.
5. Do not use pumping excessively or non stop of suction movement.

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KU.KU Duckbill Storage Milk Bottle Set- 3 Bottles

When mothers have collected excess breast milk, we suggest to change a new storage milk bottle to collect breast milk in order to manage the amount of breast milk easily. KU.KU Duckbill Storage Milk Bottle Set- 3 Bottles are designed for mothers to storage more breast milk.

- The milk bottle can use standard nipple.
- The milk bottle can be placed on the breast pump to used as the storage milk bottle.
- Put on the nipple and cover to be used as the feeding bottle.
- Foodstuff level PP material can be sterilized by high temperature.
- It can be repeat for use which is money saving and environment friendly.

KU-5924

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4 712702 870510