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- 1.Before using the product, read the manual carefully, examine the breast pump and all accessories for damage, and make sure all parts are assembled properly.
- 2.When using the adaptor, make sure it does not contact water.
- 3.The silicone tube should not be twisted or tangled.
- 4.Before each use, examine the breast pump (including the power adaptor) for visible damage. If the power adaptor is damaged, does not work properly, has been dropped, or has been immersed in water, stop using the breast pump.
- 5.Do not expose the breast pump (including the power adaptor) to high temperatures or the sun directly.
- 6.When you want to stop use, turn off the power first.
- 7.Do not tilt or overfill the milk bottle (do not go over the maximum scale) for use. If the bottle is tilted, turn off the power, hold the bottle upright to prevent spills, and restart the power to continue pumping milk.
- 8.Do not try to open or repair the motor of the breast pump by yourself.
- 9.Before each use, clean and sterilize the necessary parts.
- 10.If you just gave birth and produce less milk, or have mastitis, retracted nipples, or flat nipples, follow your doctor or nurse's instructions for using the product.
- 11.Before connecting the breast pump with the power supply, make sure the local voltage matches the requirements of the power adaptor.

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◎Battery Installation:

- 1.Remove the battery cover on the back of the motor unit.
- 2.Use four AA alkaline batteries (alkaline batteries only).
When installing batteries, make sure the positive (+) poles and negative (-) poles of the batteries are consistent with the symbols in the battery compartment.
- 3.Replace the battery cover.
 - * If you will not use it again for a long time, remove the batteries.
 - * If using batteries, when the power indicator light turns from blue to red or the suction turns weak, replace batteries with a set of new batteries to return to normal operation.
 - * If using batteries, the suction strength will gradually decrease depending on the remaining battery power.

◎Power Adaptor Installation:

- 1.Insert the DC connector of the AC power adaptor into the DC jack on the back of the motor unit.
- 2.Plug the AC power adaptor in the power outlet.
 - * The product comes with an AC power adaptor. Do not use other power adaptors.
 - * Insert the DC connector of the adaptor into the motor unit of the breast pump, then plug the adaptor into the outlet.
 - * The product works the best with the AC power adaptor for power supply.

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◎Cannot start:

- 1.Using batteries
 - A.Check if the batteries are installed in the correct polarities.
 - B.Check if the battery contacts are closely connected.
 - C.If the power indicator light turns from blue to red, replace the batteries with a new set of AA alkaline batteries.
- 2.Using the AC power adapter:
 - A.Check if the adaptor plug is correctly inserted into the DC jack on the motor unit.

◎Cannot extract breast milk:

- 1.Make sure all parts are correctly assembled and tightened.
- 2.Check if the battery power is normal (Note: the suction strength of the breast pump weakens when the battery power decreases.).
- 3.The pump shield should closely surround the breast to stop air leaking in.
- 4.The milk bottle should not be tilted during pumping. If it's tilted, turn off the power, and hold the milk bottle upright before you continue to pump.

◎Insufficient Suction Strength:

- 1.When the power indicator light turns red or the suction strength weakens, it means the battery power is low. Replace the batteries with a new set of AA alkaline batteries or use an AC adaptor to change the suction strength of the breast pump.
- 2.Check if the pump shield closely surrounds the breast and all parts are properly assembled.
 - *Replace any broken or old silicone parts with a new ones.

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為了您的安全，請您務必花費幾分鐘的時間，
在充分了解本說明書之後，請按照內容做正確的使用。
In order to protect your safety, please spend some times to read thought
this instruction and operate the device with the direction before use.

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1.The unique massage system of the electric mini breast pump with natural feelings stimulates the mammary glands to produce prolactin that increases breast milk production. It simulates a baby's sucking rhythm (suck, pause, and release) that makes expressing breast milk feel like breastfeeding babies and rapidly empties the breast. It is the most simple and efficient way to increase your breast milk.
2.It's compact, light, portable, comfortable, and quiet.
3.After three minutes in standby mode, the motor automatically shuts down, which is safe and saves energy.

◎Comfortable Breast Pump:

The pattern of the stimulation mode (first two levels)/expression mode (next four levels) of the breast pump is the most simple and efficient way to increase breast milk.
Stimulation Mode: SPA massaging the breast to stimulate milk production.
Expression Mode: simulating a baby's sucking rhythm (suck, pause, and release) to cyclically and efficiently express milk.

◎Super Pump Power:

The motor directly outputs 300mmHg and more strength. There are six levels of settings in stimulation and expression modes. Select the most comfortable expression based on personal preferences.

◎Ultra Quiet:

It is designed with an ultra quiet motor to avoid waking the sleeping baby while operating.

◎Automatic Memory:

It automatically memorizes the previous frequency and never needs a reset.

◎SPA Massage:

Soft SPA massage system helps to stimulate milk production.

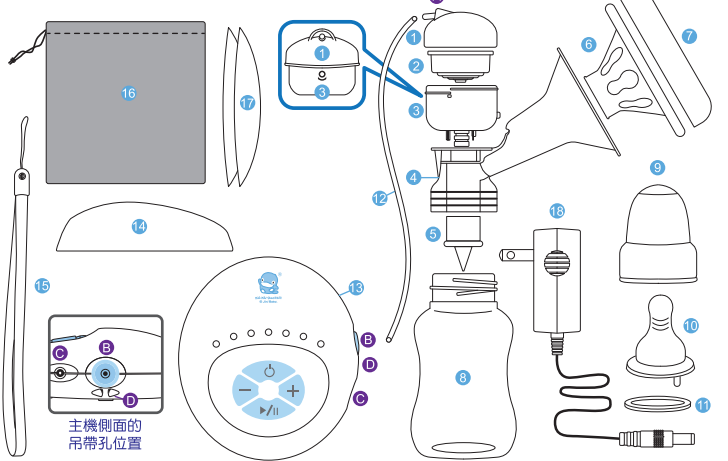
◎Soft Silicone Massage Cushion:

The soft silicone massage cushion massages the breast to reduce the discomfort and stimulate mammary glands to secrete more milk.

◎Air Hole Design:

Designed air hole on shield help breast milk going to bottle smoothly and fast.

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- 1.Insert the silicone Cushion ⑥ into the pump Shield ④.
- 2.Assemble silicone diaphragm ② into vacuum base ③, turn the cap ① to right to lock it, align A on cap with the symbol ㊟ on vacuum base ③.
- 3.Put assembled vacuum base ③ into funnel ④ from up to down.
- 4.Put anti-back flow device ⑤ into funnel ④ from bottom to top.
- 5.Screw Milk Bottle ⑧ onto assembled Shield ④.
- 6.Connect the end of silicone Tube ⑫ onto A on the Anti-backflow Device ① and socket B of the Motor Unit ⑬ respectively. At last, plug the Power Adaptor ⑭ into the socket C of the Motor Unit ⑬. (Transformer, universal voltage 100V ~ 240V)
- 7.The Neck Strap ⑮ can be attached to the hole D on the side of the motor unit. A Storage Bag ⑯ is also included for moms' convenience to store and carry the device.
- 8.Breast Pads ⑰ is attached.

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- 1.Closely attach massage cushion and pump shield to the breast without gaps to avoid a loose suction.
- 2.Turn on: press ㊟ the power button. The blue LED light will turn on, and the six-level LED lights (white) will flash.
- 3.Level Settings: stimulation (first two levels) mode and expression (next four levels) mode. You may set the stimulation/expression frequency by pressing + the plus button to increase the suction strength or - the minus button to decrease the suction strength based on personal preferences.(There are six levels of suction strength for the product. The number of indicator lights that are on represents the suction strength.)
◎You don't need to choose the largest suction strength but a level that makes you feel comfortable.

- 4.Start/Pause: A. Start: pressing ▶/|| to start the motor unit for operation.
B. Pause: pressing ▶/|| to pause the suction. The LED lights will flash.
C. Resume: pressing ▶/|| to resume the operation. The LED lights will turn on.
- 5.When the pumping is finished, press ㊟ the power button to turn off the breast pump and remove it from your breast.
※After the breast pump is turned off, the micro computer will automatically memorize the last settings for the next use.

Do not directly remove the pump shield from your breast when the breast pump is in a vacuum state.Turn off the power, use a finger to lift the massage cushion from your breast to relieve the enclosed vacuum, and then remove the breast pump from your breast.

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The steps to cleaning and sterilization before using it for the first time and after each use of your breast pump:

- 1.Disassemble all parts that have contacted the breast milk (except the motor unit and adaptor).
- 2.Wash all parts with warm water and then sterilize them (* except the motor unit, battery cover, stand, and power adaptor).
- 3.Air-dry all sterilized parts in a cool, dry place.
- 4.Do not put the motor unit and power adaptor into water, nor wash with water.
- 5.When disassembling or washing parts, be careful not to lose smaller parts since it may result in reassembly and machine operating problems.
- 6.When connecting the silicone tube, dry before use.

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