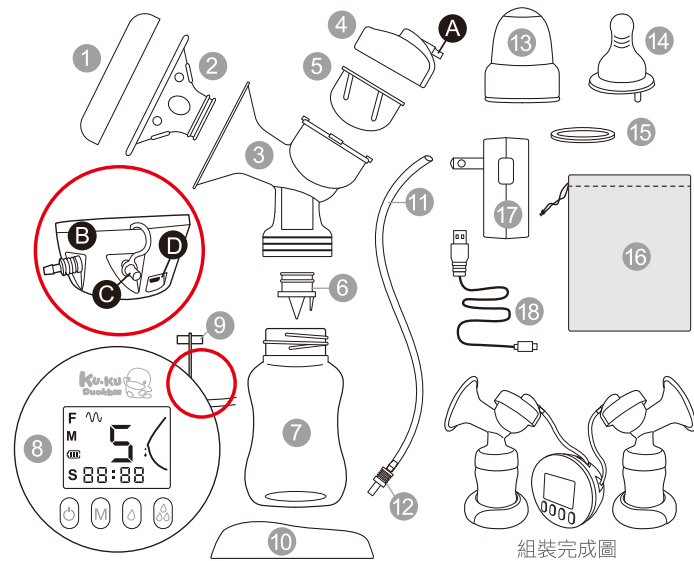
安裝說明 | Instructions

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1. Push the Massage cushion **2** into the Breast shield **3** as deep as possible. Make sure the massage cushion is properly attached without any leak.
2. Insert Silicone diaphragm **5** onto the upper Breast shield **3**. Screw the Top cover **4** properly.
3. Install Spout valve **6** onto lower Breast shield **3**.
4. Screw Bottle **7** onto lower Breast shield.
5. Connect Air tube **11** onto Top cover **A** of the Breast shield and air tube socket **B**, **C** on Motor unit **8** with connector **12**.
6. Plug the electric adaptor **17** into the electric adaptor socket **D** of the motor unit.
7. To use single Pumpset: Plug the Stopper **9** into the air tube sockets **C** of the motor unit.

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- | | | | |
|---|---------------------------|------------|----|
| ① | 防塵蓋 Cover | 塑膠P.P | 2入 |
| ② | 矽膠墊片 Massage cushion | 矽膠Silicone | 2入 |
| ③ | 吸乳罩 Breast shield | 塑膠P.P | 2入 |
| ④ | 氣管蓋 Top cap | 塑膠P.P | 2入 |
| ⑤ | 矽膠碗 Diaphragm | 矽膠Silicone | 2入 |
| ⑥ | 矽膠鴨嘴閥 Spout valve | 矽膠Silicone | 4入 |
| ⑦ | 奶 瓶 Milk bottle | 塑膠PPSU | 2入 |
| ⑧ | 主 機 Motor Unit | 塑膠ABS | 1入 |
| ⑨ | 矽膠塞 Stopper | 矽膠Silicone | 1入 |
| ⑩ | 底 座 Bottle stand | 塑膠P.P | 2入 |
| ⑪ | 矽膠管 Tube | 矽膠Silicone | 2入 |
| ⑫ | 連接頭 Tube Connector | 塑膠P.P | 2入 |
| ⑬ | 奶瓶蓋 Cap set | 塑膠P.P | 2入 |
| ⑭ | 奶 嘴 Nipple | 矽膠Silicone | 2入 |
| ⑮ | 儲乳蓋 Sealing disc | 矽膠Silicone | 2入 |
| ⑯ | 收納袋 Storage Bag | 尼龍Nylon | 1入 |
| ⑰ | 變壓器 Power Adaptor | | 1入 |
| ⑱ | USB充電線 USB charging cable | | 1入 |

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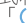
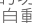
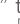
1. There is USB port where you can just use your computer or power supply bank when you are out. Adaptor when you are at home.
2. Only use the power adaptor supplied with the kuku duckbill breast pump. Never use the breast pump adapter with any other equipment.
3. Insert the AC adapter plug firmly into the socket on the motor, and then insert the AC adapter into the outlet.
4. Use directly power plug for best pumping result.
5. Equipped with internal rechargeable lithium ion battery (approximately three hours of use per full charge).
6. If the breast pump is not used for a long time, it is advisable to fully charge and store the unit. Charge the breast pump at least once every three months during the periods.

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1. Use the heat towel to warm your breasts. Massage your breasts with fingers 3~5 minutes to stimulate lactation and makes you comfortable.
2. Turn on the Power button to start function.
3. Massage mode and Pumping mode:
After switching on the device by 「」 button, it automatically starts with 2 minutes of "Massage mode" and then "Pump mode". After "massage mode" you will have "pump mode" afterwards.
 - Pressing 「」 button or 「」 button, breast pump will change from "Massage mode" to "Pump mode" manually.
 - When you press the 「」 button, the breast pump will back to the "Massage Mode".
 - The device will remember your last setting of suction, will automatically change to the last level you set.
4. Adjust the Lever (suction strength):
Press the 「」 to adjust the level. The vacuum level will appear on the display screen ("1" indicates the lowest level and "12" indicates the highest level)
 - ※ Adjust vacuum level accordingly to achieve maximum comfort. When you need to release from milk engorgement, you can try highest level of suction.
 Switch between "Pump mode" and "Comfort pump mode"
5. Basic suction and Massage suction:
Switch to massage suction :Press the 「」 and 「」 keys at the same time until 「」 twinkles on the display.
Back to basic suction to press the 「」 and 「」 keys at the same time until 「」 does not twinkle on the display.
※ Massage suction is that soft massage before suction every time.
6. Frequency:
In the suction, short press 「」 button ,you can adjust the Frequency 「」 「」 「」 「」 to change the speed.
When you have finished pumping, press the 「」 button and
7. unplug the adapter. (More than 3 seconds)
 - ※Breast pump switches off automatically if it has been running for 70 minutes with no interruptions.

The keys				
短按 Short press	頻率切換 Frequency Switch	按摩 Massage	吸力減少 Level down	吸力增加 Level up
長按 Long press (More than 3 seconds)	開關機 Turn on/off		同時短按 一般/舒適 模式切換 At the same time short press, Basic Suction switch to Massage Suction	

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1. Please read this instruction carefully before using the product. Assembly or disassemble the accessories carefully.
2. Never use the breast pump in the damp place or someplace near water. If the main device drops into the water, do not touch it and please unplug with dry tools.
3. Inspect all appropriate pump components before each use. Never operate an electrical device if it has a damaged cord, plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. If damage is found, immediately discontinue use of power adaptor.
4. Make sure tubing is not kinked or pinched while pumping.
5. Do not store the product under direct sunlight, close to direct heat sources.
6. Do not fill higher than the maximum level indicated on the bottle. **IMPORTANT:** Sit upright and do not tilt the bottle while pumping, it should be kept upright at all times to avoid milk flowing back up through the air tube. If this happens you must turn the breast pump off immediately as it could damage the motor.
7. Do not attempt to modify or disassemble the product.
8. During charge or storage, if the product is over heated, the case is discolored, it may lead to battery fluid leakage, swelling of the case, smoke, or fire, please stop using immediately.
9. Apply hot towel to the breast to promote expressing.
10. If you just gave birth and produce less milk, or have mastitis, retracted nipples, or flat nipples, follow your doctor or breastfeeding specialist instructions for using the product.
11. If you can express only minimal or no milk or if expression is painful, stop using it immediately and consult a doctor.

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SMART Electronic Double Breast Pump

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In order to protect your safety, please spend some times to read thought this instruction and operate the device with the direction before use.

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- Smart adjustment**
With the 3 speeds, 12 levels of suction setting, allow to switch between the basic suction and massage suction, moms are able to adjust speed & intensity according to their personal comfort levels.
- Lactation massage**
Once turned on, it will automatically start its 2-minute breast massage in order to stimulate the secretion of breast milk. While pumping, you can switch to comfortable pumping mode to massage and pump at the same time so as to decrease the level of blocked ducts.
- Smart memory function**
It automatically memorizes your previous settings and saves you from the trouble of readjusting over and over again.
- USB charging support**
Input 100~240 VAC, 50-60HZ. There is USB port where you can just use your computer or power supply bank when you are out. Equipped with internal rechargeable battery (approximately 3 hours of use per full charge)
- Noctilucent and mute**
The device runs quietly and comes with a noctilucent LCD display including keys which allow you to pump at night without turning on the lights and waking up the babies.
- Skin-friendly cups**
The petal-shaped silicone cups help you release the uneasiness from pumping and continuously massage your breasts, making you feel comfortable and pump better.
- Light-weight and convenience**
Being weighed only 216g, the device is light-weight and easy to carry around. Also, it comes with top quality golden PPSU feeding bottles which give you the best level of health and care.

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Can't switching on

Power supply didn't plug in the socket. Check the electricity of the battery.

No suction

1. Make sure all connections are secure.
2. Check the electricity of the battery.
3. Ensure the massage cushion fit on the breast.
4. The valve or tube has been broken during sterilizing.

Weak suction

1. Top cover is leaking, check the massage cushion and top cover (firmly fit to hear a click).
2. Valve is leaking.
3. The second air tube socket of motor unit remains closed – by single pumping.

Milk back flow

1. The milk flow is large and choose 8 or 9 level.
2. There is water in the tube, Dry or change the tubes.
3. Bottle is too small

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The steps to cleaning and sterilization before using it for the first time and after each use of your breast pump:

1. Disassemble the breast pump completely before cleaning and disinfecting.
2. The motor and the adapter can not be cleaned and disinfected.
3. The tube connector shall not be disinfected.

Contact with breast milk: Massage cushion, breast shield, valve, bottle, nipple, screw ring, please clean immediately after every use.

Not contact with breast milk: Cover, top cover, tube, bottle stand, connector need clean every week.
Main device can be cleaned by wet tissue.

**Attention: ※Clean the valve carefully not to break the tip.
※Never clean with solvents or harsh chemicals.**

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Please clean all accessories contact with breast milk by special detergent and sterilize.

1. **Disinfection by boiling:** Put cleaned parts in boiling water for 5 minutes.
• Please put the parts that could be sterilized in boiling water, pay attention not to letting them contact the pot wall, to preventing deformation or damage.
• Excessive boiling will shorten life of the parts. **(except the motor unit, air tube)**
2. **Disinfection by microwave:** Please use a special microwave container for disinfection.
• Please operate in accordance with the microwave operation instructions.
• Please make sure that there is water in the container while heating it.
Steam sterilizers for babies are recommended.
3. **Other methods:**
Never use dry heat, UV or a dish washer/dryer to disinfect, or the parts maybe damaged or deformed.
**※Air-dry all sterilized parts in a cool, dry place.
※When connecting the silicone tube, dry before use.**

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1. Once finished pumping, remove the bottle and place the screw caps for breast milk storage.
2. Place the nipple and screw caps on the bottle, then it can be used for feeding.
3. If you express more than once a day, store milk in the refrigerator for up to 24 hours. Thaw milk by standing the bottle in warm water until it has reached body temperature.(Test the temperature by dropping some on your wrist before feeding baby)
※Recommend store breast milk in kuku duckbill bottles with screw caps or breast milk storage bag.
4. You can freeze and/or refrigerate your expressed breast milk, see storage guide below:
• store it at room temperature: for up to 8 hours (at no warmer than 77°F, or 25°C)
• store it in the refrigerator: for up to 5 days at 32°~39°F (0°~4°C)
• store it in the freezer for 3 to 6 months in a freezer that's kept at 0°F (-18°C). Store the milk in the back of the freezer, not in the door.
5. To warm milk: heat water in a small container (no warmer than 140°F, or 60°C), then place frozen milk in the water to warm or use a bottle warmer.
NEVER microwave breast milk or heat it directly on the stove.
※Recommend warm milk in the kuku duckbill warmer, easy and fast.

KU5924
酷咕鴨儲乳瓶
Storage Milk Bottle Set

KU5447
母乳/副食品儲存袋
Milk & baby food
storage bag

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JINI BABY INTERNATIONAL CO.,LTD.